



# SaniiSwab™

by nevoo

## Nasal Cleansing Revolutionized

The first 1-minute, 2-step, disposable surgically-proven treatment.



Protect your patients—and your bottom line—from costly HAIs



Helps reduce risk of infection and contaminant spread in your medical facility



For daily use by patients, hospital personnel and visitors

## DO SOMETHING AWESOME FOR YOUR NOSE



Clean away debris

Swipe away allergens

Wipe out germs

**NOSEY ABOUT YOUR NOSE?**  
SCAN HERE  
[saniiswab.us](http://saniiswab.us)




### SaniiSwab daily, 2-step cleansing system can help reduce HAIs

Previous sanitation protocols focused on the physical environment. Today, in light of the Covid pandemic, **there is a greater emphasis on both hand and nasal sanitation.**


Because the **nasal cavity is a breeding ground for germs, viruses and bacteria**, daily nasal cleaning has been proven to be effective in reducing HAIs.

**Daily nasal cleaning for patients, staff and visitors** assists in reducing infiltration of contaminants in your facility. **SaniiSwab** makes this daily process easy, fast and cost effective.

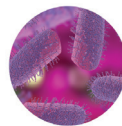

### BACTERIA

- Staphylococcus aureus
- Staphylococcus methicillin resistant (MRSA)
- Staphylococcus epidermidis
- Staphylococcus pneumonia
- Enterococcus
- Clostridium difficile
- E. coli
- Haemophilus influenzae
- Klebsiella pneumoniae

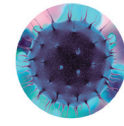


### VIRUSES

- Influenza A and B
- RSV
- Parainfluenza
- Adenovirus
- Rhinovirus
- Enterovirus
- Herpes simplex
- Coronavirus
- Covid-19
- SARS

## What makes SaniiSwab unique?

### SaniiSwab follows strict, 2-step, surgical protocol

Patented columellar guard and anatomically designed swab length ensure correct placement in the lower nasal skin—**avoiding nasal mucosa.**

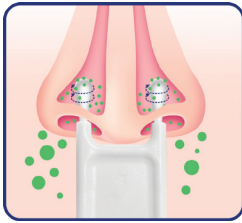


#### Step 1: **CLEAN**

Open easy-tear packet.

Place pre-saturated cleaning swabs (one for each nostril to avoid cross-contamination) against nasal skin and rotate 7 times.

This removes secretions, mucous and debris to improve **SaniiClean™** (Step 2) effectiveness. Discard cleaning swab. **Never re-use**

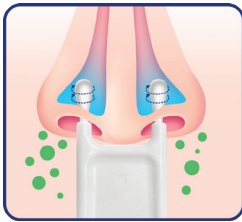

#### Step 2: **SaniiClean™**

Open easy-tear packet.

Repeat Step 1 action with pre-saturated sanitizing swabs to help reduce germs, bacteria and allergens.

Discard sanitizing swab. **Never re-use.**

### SaniiSwab's patented double-swab design eliminates cross- and recontamination

**Cross-contamination** occurs when a utensil is used twice to clean or sanitize two different areas of skin. **Recontamination** occurs when a utensil is used twice on the same area. PolleySwab is designed to avoid both.

**SaniiSwab's** pre-saturated, single-use double swab allows only one swab to enter each nostril simultaneously and is immediately disposed of after the completion of each step.



### WHY THIS WORKS SURGICALLY PROVEN

"Your **nose** is the main entry point for infection—which is why **daily nasal care** is as **important as washing your hands.** My 2-step cleaning and sanitizing method—used in over 2000 nasal procedures with ZERO infections—is now available for anytime, anywhere use... **PolleySwab™.**"



JW Polley, MD,  
Craniomaxillofacial Surgeon  
and developer of PolleySwab™